

How To Use Your Chuck Brush Chuck Brush Instructions

- 1: **Lubricate** Chuck Mechanism with **1 drop of Lubricant.** *(using Health Canada/FDA Approved Mineral Oil Lubricant)*
- 2: Insert chuck brush into chuck mechanism.
- 3: Turn chuck brush in a **circular motion, back and forth** for 5 seconds.
- 4: **Lubricate** chuck mechanism with **1 drop of Lubricant, repeating Step A**
- 4: Run Handpiece **(with bur or shipping post)** for 5 seconds to expel debris.



Chuck Brush Is Reusable - Please Ensure Suitable Cleaning of Brush after Every Use.

More Maintenance Information Available: www.mcshandpiece.com/maintenance

TF: (888) 575-8544

TXT: (604) 373 6747

EM: info@mcshandpiece.com

www.mcshandpiece.com

Office Hours: 8am - 2pm, Monday - Friday (PST)

Office Closures: Please Check Our Website for Details!

www.mcshandpiece.com/business-hours

MCS www.mcshandpiece.com
is Mobile Friendly!

 facebook.com/mcsandpiece

 twitter.com/mcshandpiece

 instagram.com/mcshandpiece

 pinterest.com/mcshandpiece