

- 1: **Insert** a Bur or Shipping Post into the **Chuck**, & **Attach** the **Corresponding Coupling** to the handpiece.
- 2: Using the **Small of the 2 Largest Holes on the Coupling** attachment, **Spray 2 Second Spray or 4 Drops of Lubricant** into the handpiece. *(using Health Canada/FDA Approved Mineral Oil Lubricant)*
- 3: Using your **Index Finger & Thumb**, hold the **Bur or Shipping Post & Spin the Turbine**. The bearings should come loose & will allow the turbine to spin.
- 4: Once bearings are spinning freely, attach to a chair and purge the handpiece. You may find black oil coming from the chuck. This is normal. If black oil continues to come out, **Repeat Step 2 Until Black Oil Become Clear**.
- 5: Sterilize handpiece as per Manufacturer's Instructions.

More Maintenance Information Available: www.mcshandpiece.com/maintenance

TF: (888) 575-8544

TXT: (604) 373 6747

EM: info@mcshandpiece.com

www.mcshandpiece.com

Office Hours: 8am - 2pm, Monday - Friday (PST)

Office Closures: Please Check Our Website for Details!

www.mcshandpiece.com/business-hours